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
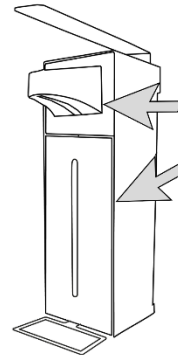
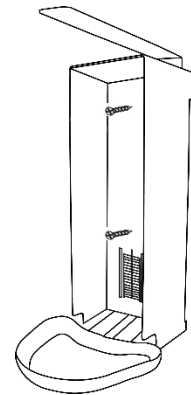
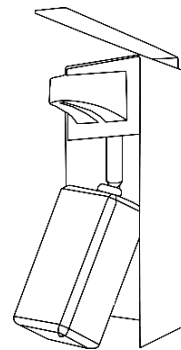
(Abb. 6) / (Fig.6)

## Description

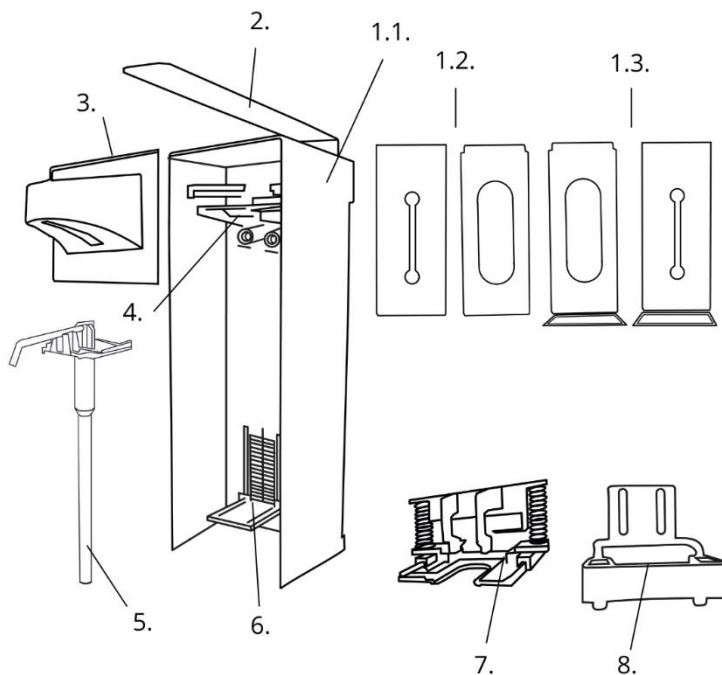Manual arm lever dispenser made of stainless steel in upright design for EURO bottles (500-1000ml) of all soap and disinfectant manufacturers. Two versions are offered: The **SD6000** is with optional four different bottle end panels - 2 window sizes with/without integrated metal drip tray. The **SD6010**, without or with a drip tray made of plastic in different colors, optional two different bottoms of bottles, differentiated by the size of the window.

- In addition to wall mounting, the versions with drip tray can also be used as a stand-alone unit.
- Both versions offer further configuration options
- The operating levers are available in solid stainless steel or wire or plastic in various colors and lengths- Piston stroke pump made of plastic (jet, spray, foam)
- Adapter for hygiene containers/pump bottles or Ophardt pumps/HyBag


Schematic diagram SD6000 (Fig. 1)

## Schematic diagram

1. Metal housing
- 1.1 Housing body with integrated wall mounting plate
- 1.2 Bottle cover with viewing window without drip tray
- 1.3 Bottle cover with viewing window and drip tray
2. Arm lever
3. Pump cover
4. Pump holder
5. Disposable pump
6. Bottle lock
7. Adapter für Ophardt Pumpen/HyBag
8. Adapter für Hygienegebinde
9. Drip tray

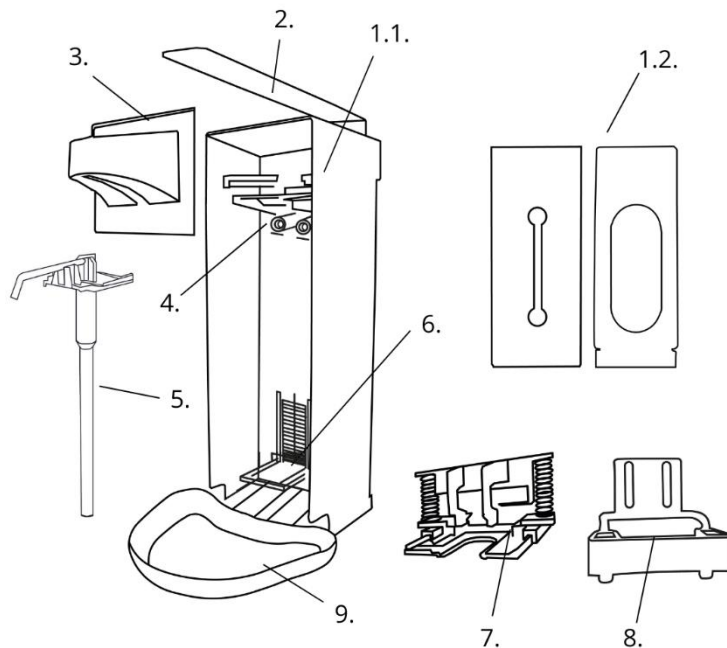Schematic diagram SD6010 (Fig. 2)

### Note before mounting:

Unpack the box, remove the plastic packaging of the dispenser, and make sure that the following parts are included (Fig. 1 and Fig. 2):

1 x Dispenser with plastic disposable pump (Fig. 6)

1 x Operating lever mounted on the dispenser

1 x Bag with two set screws and dowels each, short pump suction tube for 500ml bottles

1 x Operating instructions de | en optional with stickers disinfection | soap

**Optional chargeable accessories:** lockable bottle end panel with/without integrated drip tray, colored plastic drip tray and arm lever (just for SD6010), Mounting set (SD2005-78): 2 x hook with 4 x screws and 1 x glue, Foam/spray pump, Adapter for hygienic packaging or for Ophardt pumps/HyBag-, Stainless steel bent wire arm lever and XXL stainless steel arm lever.

## 1 Installation / Operation

### 1 a Spender komplettieren

- The dispenser is supplied with the operating lever disassembled.
- Push the operating lever into the dispenser's back until it sounds like it is engaged.  
The dispenser can be used as a stand-alone unit or mounted on the wall.

### 1 b Dispenser assembly (Fig. 4)

A drill and a 6mm drill bit as well as a screwdriver or cordless screwdriver are required for wall mounting. A drilling template is printed on the lid of the dispenser packaging. Select a suitable installation location and height. Height recommendation: The upper edge of the dispenser should be approx. 130 cm above the floor.

- Fasten the carcass to the wall with dowels and screws or the optional adhesive mounting set.

## 2 Implementing

- Slide the full bottle from below over the dispenser pump and insert it into the housing (Fig. 5); the dispenser is prepared for operation with 1000 ml bottles, if 500 ml bottles are used, the pump suction tube must be exchanged for the shorter one supplied. If necessary, close with optional bottle end cap and attach drip tray.
- When using for the first time, press the operating lever down several times as far as it will go until liquid emerges from the pump nozzle; when using viscous gels, soaps or lotions, the lever must be pressed down firmly up to 10 times before solution emerges! In case of problems, first aspirate thin-bodied liquid (e.g. water), then thick-bodied liquid.

## 3 Bottle or pump exchange and dosing

- Open the dispenser, pull the empty bottle down to the front and remove it, loosen the pump cover from the holder. Remove the pump. When doing so, loosen the locking device of the pump on the wall side of the body. Dispose of the pump.
- Insert new pump and proceed as described under 2.
- The dispenser pump is set to the maximum dosing quantity of 1.8 ml per stroke on delivery.

- When using viscous media, please pump-up new pumps only with the maximum setting (dosing quantity 1.8 ml)!
- By moving the dispensing fork to the right or left, the dispensing volume can be reduced to 1.2 ml or 0.8 ml per stroke (Fig. 6).

## 4 Operation with adapter for Ophardt DHP pumps or HyBag

- Remove bottle and existing pump (s. 3).
- Push the adapter into the guide rail from the front like a pump until the lugs of the spring holders abut the guide rail on both sides.
- Slightly compress the spring on both sides to allow the nose to pass the side of the guide rail.
- The adapter can now be easily pushed further in the guide rails to the rear wall until it snaps into place on the mounting lug of the rear wall like the pump
- Insert the Ophardt DHP pump and bottle, close the bottle cover if necessary.

## 5 Operation with adapter for hygienic packages

- Open dispenser, remove bottle and existing pump (s. 3).
- Mount the adapter below the pump bracket at the two mounting points using the two screws provided.
- Insert the bottle neck of the hygiene container into the rubber holder. Check the function of the arm lever and adjust the adapter vertically, if necessary, by slightly opening the screws and moving the adapter. Fix the screws. Pull the bottle locking plate to the bottle base and lock it in place.

## 6 Cleaning the dispenser | Hygiene recommendations

- The dispenser is maintenance-free, but the advantages of dispensers made of stainless steel are guaranteed only if the dispensers are cleaned and disinfected regularly. Insufficient dispenser hygiene can lead to germ colonization of the dispenser system. Disinfection or soap residues in the pump or on the nozzle can also cause problems. Therefore, the dispenser should be cleaned and possibly disinfected regularly according to the recommendations for infection prevention and hygiene.
- For this purpose, clean the housing with a mild detergent only; if desired, treat with surface disinfectant
- The housing can be cleaned in a suitable dishwasher.
- The **Robert Koch Institute** expressly recommends the use of disposable bottles or hygiene containers for washing and care products. The German Medicines Act applies to hand disinfectants: empty bottles may only be refilled by a pharmacy under aseptic conditions
- For hygienic reasons, the dispenser pump is designed as a disposable pump, which **should be replaced at regular intervals** with the bottle change at least **every 3-4 months**. The pumps must not be reprocessed but **must be renewed**. The pumps are available as jet, spray, or foam version.

## 7. Accessories and Spare Parts

SD2005-10	Disposable jet pump for 500ml bottles
SD2010-10	Disposable jet pump for 1.000ml bottles
SD2005-11	Disposable spray pump for 500ml bottles
SD2010-11	Plastic disposable pump spray for 1.000ml bottles
SD2005-12	Plastic disposable pump foam for 500ml bottles
SD2010-12	Plastic disposable pump foam for 1000ml bottles
SD2005-20-w	White drip tray
SD2005-20-b	Blue drip tray
SD2005-20-g	Green drip tray
SD2005-20-p	Pink drip tray
SD4005-45	Bottle lock
SD2005-60	Adapter for Ophardt pumps
SD2015-60	Adapter for hygiene packagings
SD2005-78	Adhesive mounting set Tesa Nie wieder bohren
SD6000-30	Bottle cover with large oval window for SD6000
SD6000-31	Bottle cover with narrow vertical viewing for SD6000
SD6010-30	Bottle cover with large oval window for SD6010
SD6010-31	Bottle cover with narrow oval window for SD6010
SD6000-32	Bottle cover with narrow oval window for SD6010
SD6000-33	Bottle cover with large oval window with integrated stainless steel drip tray Bottle cover with narrow oval window with integrated stainless steel drip tray
SD6000-40-d	Colored plastic cover for pump (gray)
SD6000-90	Arm lever from bent wire
SD6000-91	XXL arm lever solid stainless steel
SD2005-90	Arm lever short plastic (white, blue, green or pink)
SD2010-90	Arm lever short plastic (white, blue, green or pink)
SD6000-55	Arm lever long plastic (white, blue, green or pink) Lock and key hinges